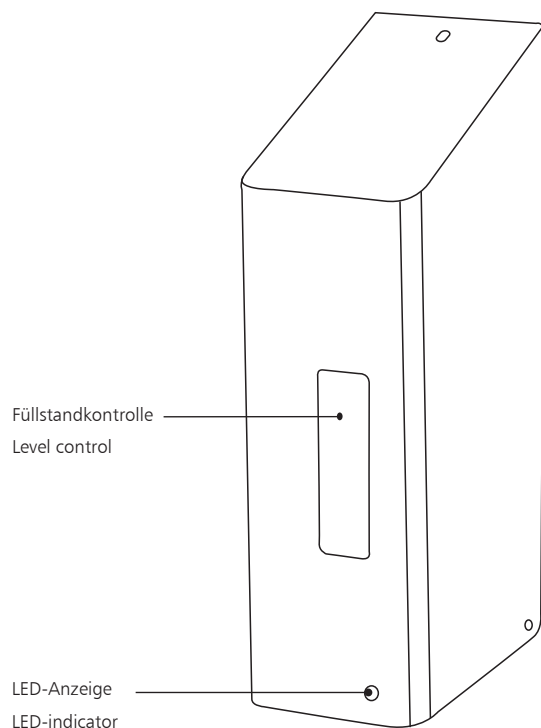
SanTRAL® NSU 11 Touchless

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E 1. Scope of delivery

- Dispenser
- Fixing set (4 screws and plugs) and key
- Instruction manual
- Batteries (4 x Typ D)

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E 2. Intended use

Automatic dosed dispenser for liquid soap, foam soap or alcoholic disinfectants (depending on pump version). Refill volume max. 1200 ml. Only for use in enclosed rooms without risk of explosion in non-condensed environment. **Not suitable for use at swimming pools.**

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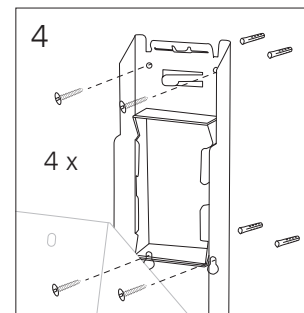
E 3. Opening the device


To open the device, unlock the cover and push vertically upwards until you feel a resistance. Now fold the cover down into the end position. The safety switch prevents injuries and incorrect operation by inadvertent activation of the dispenser when the cover is open. **Warning! Do not reach into the pump unit!**

E 4. Installation

Open the device and actuate the rear unlocking tab and remove the battery compartment cover by lifting upwards (see Fig. 5). Set down the cover in a secure and dry place. The ergonomically optimum installation height for safe and comfortable operation is 110cm, measured from the floor to the base of the dispenser housing. 20cm of space must be left free beneath the dispenser.

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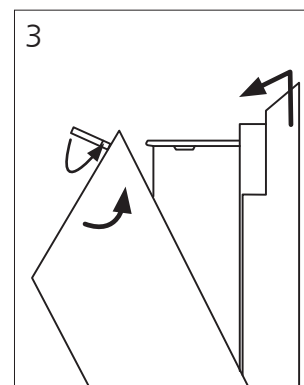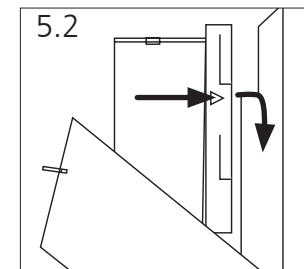
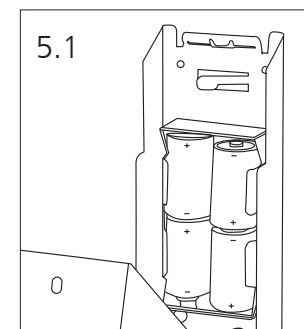
Important!
To ensure safe operation, the dispenser must be mounted to the provided fixing apertures on the wall using 4 screws. Check the suitability of the wall beforehand! Mount the device on the wall using the enclosed fixings or other suitable fixings.

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E 5. Replacing batteries

Open the device, actuate front unlocking tab and lift out the refill container (see point 8.). Put the latter to one side in a safe place. Then actuate the rear unlocking tab and take out the battery compartment cover (metal) by lifting it up and then towards you. Set down the cover in a secure and dry place. Insert the batteries (4 x D cells) in the battery compartment on the back wall. Please note the polarity (see Fig. 5.1). Then slide the cover from the front to the bank and then down into the dispenser, thereby closing the battery compartment. An arrow on the cover show the height at which the cover is to be inserted from the front. Fix the cover again with the unlocking tab (see Fig. 5.2). Only use mono cell batteries (type D) with 1.5 V voltage. No rechargeable batteries.

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

E 6. Filling the container

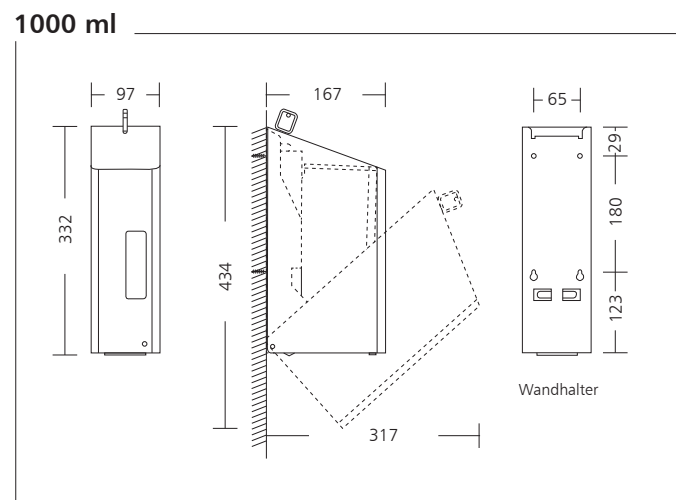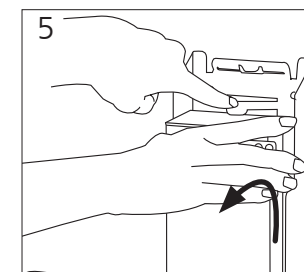Open the device. Remove the cover and fill with the cleaning agent. Alternatively, the container can be removed for filling (see Fig. 8). **Important!** Only use suitable cleaning agent in accordance with the pump type (liquid soap / foam soap / alcoholic disinfectant). Only use fluid cleaning agents without abrasives. Tests are required before new filling materials are used for the first time.

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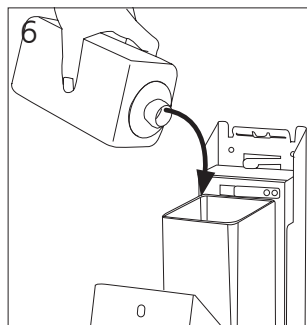
The compatibility of filling material and dispenser system must be checked prior to the use of new filling materials, in order to be able to guarantee fault-free functionality. OPHARDT hygiene assumes no liability for any damages or costs which arise due to the incompatibility of filling material and dispenser systems.

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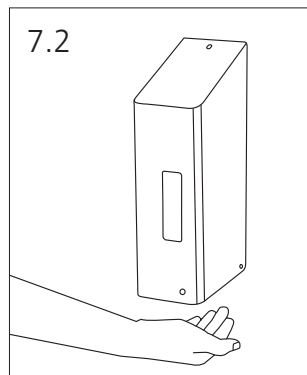
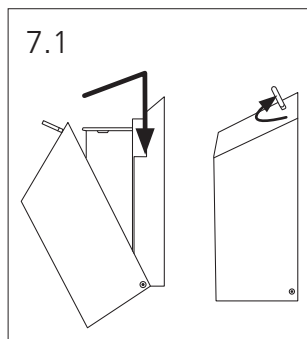
E 7. Close housing/Operating

To close the device, raise the cover slightly in order to engage it in the end position in the catch (see Fig. 7.1).

Operation: Hold your hand under the outlet (see Fig. 7.2).

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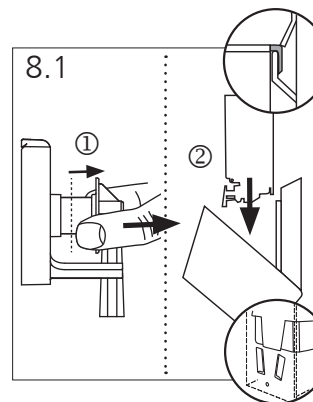
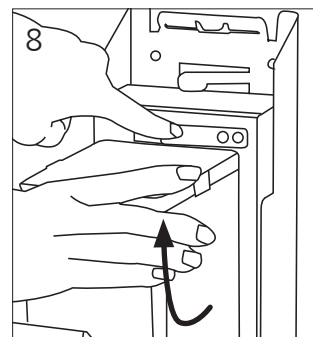
E 8. Replacing / installing container with pump

Open the device. With one hand, press back the metal springs above the container. With the other hand, pull the container gently forwards and then remove it by lifting it out (see Fig. 8).

Then, pull the pump on the refill container forwards by hand as far as it will go and insert the refill the container from above into the dispenser. Note! Make sure the container is properly seated in the dispenser. The container must be located in the guide at the bottom and must be attached by the hook at the top (see Fig. 8.1).

The locking tab on the cover must retain the top edge of the container.

Close the device. Note: Container and pump can be replaced together so that other cleaning agents (liquid soap, foam soap, alcoholic disinfectant) can be used.

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E 9. Maintenance and cleaning instructions

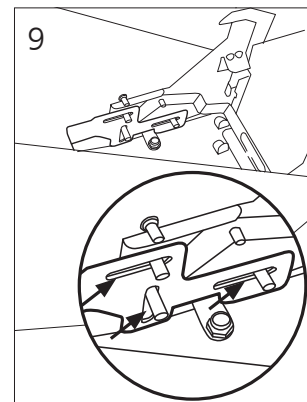
For the most part, the device requires no maintenance. However, it must be cleaned if it becomes soiled. Regular external wipe disinfection/cleaning.

Soap and disinfectant residues inside the device must be removed at regular intervals. Important! The cleaning agents must be compatible with plastics.

For fault-free operation, the king pins on both sides of the moving presser must be greased annually with Vaseline (see Fig. 9).

To open the device, unlock the cover and push vertically upwards until you feel a resistance. Now fold the cover down into the end position.

To close the device, raise the cover slightly in order to engage it in the end position in the catch. The safety switch prevents injuries and incorrect operation by inadvertent activation of the dispenser when the cover is open.

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E 10. Procedure in event of non-operation or a fault

If there is an unexpected fault, you can attempt to fix it with the aid of the instructions provided here.

No power supply:

- Ensure that new batteries have been inserted into the device
- Make sure that the polarity of the inserted batteries is correct.
- Device possibly not completely closed.

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E 11. Disposal of used electrical and electronic devices

The symbol on the product or its packaging indicates that this product should not be treated as standard household waste, but should be taken to a recycling point for electrical and electronic devices. Further information may be obtained from your municipality, the municipal disposal companies or the company from where you purchased the product.

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E 12. Statutory information on battery disposal

Batteries should not be disposed of as household waste. As a consumer, you are statutorily obliged to return used batteries. Old batteries should be surrendered to the public collection points in your municipality or to anywhere where batteries of this type are sold.

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E 13. LED Status Indicator Light

LED shines green on actuation = function OK

LED shines continuously red for 1 minute then flashes 3 times in short succession every 30 seconds = dispenser not correctly closed.

LED flashes red once every few seconds = please replace batteries.

OPHARDT HYGIENE-TECHNIK GmbH + Co. KG • Lindenau 27 • 47661 Issum • Germany

Phone +49 28 35 18-0 • Fax +49 28 35 18-77

Email customer@ophardt.com • www.ophardt.com