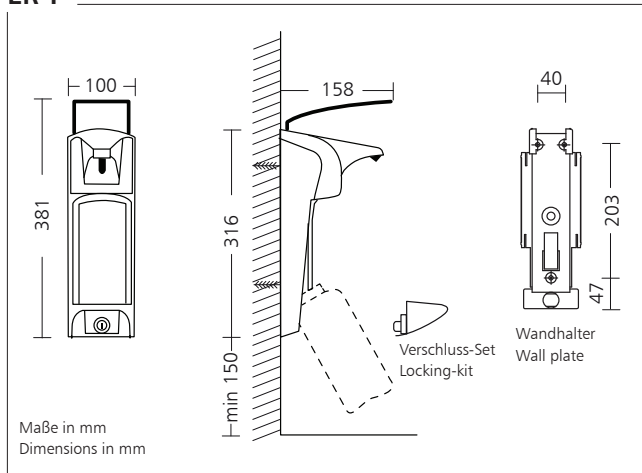
1000 ml

ER T

Technische Änderung vorbehalten. • Technical modifications reserved. • Modifications techniques réservées.

D Vor Spendermontage beachten

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E Check before installation

1. The dispenser should be fitted above a sink so that wet hands can drip.
2. Make sure you have sufficient space below the dispenser (min. 15 cm) in order to be able to remove the container.
3. Required height: 130–135 cm, from floor to top of dispenser.
4. Space of approx. 3.5 cm is needed above the wall plate for the inserting and removal of the dispenser.

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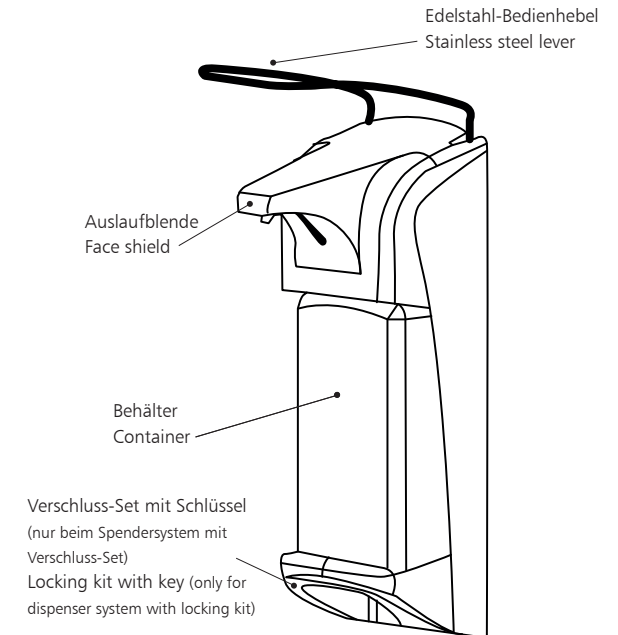
Manual cleaning: Regularly remove adherent product residual. Cleaning of the dispenser housing and the pump with warm water. Rinsing the pump with warm water. Disinfection of shroud is possible with appropriate surface disinfectants.

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The compatibility of filling material and dispenser system must be checked prior to the use of new filling materials, in order to be able to guarantee fault-free functionality. OPHARDT hygiene assumes no liability for any damages or costs which arise due to the incompatibility of filling material and dispenser systems.

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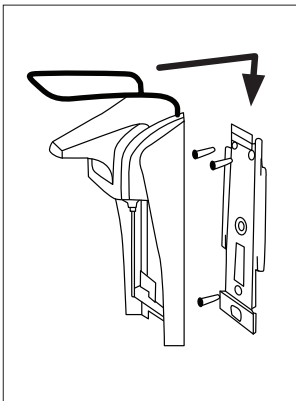
E Scope of delivery

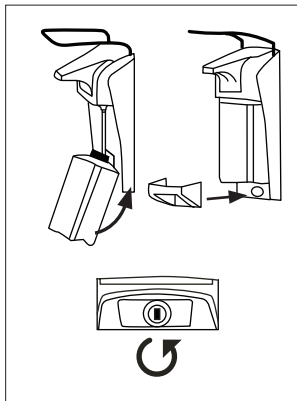
- ER T device
- Fixing set
- Operating instructions
- Locking kit (only for dispenser system with locking kit)

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E Intended use

The device is intended solely for dispensing liquid soaps and disinfectants.

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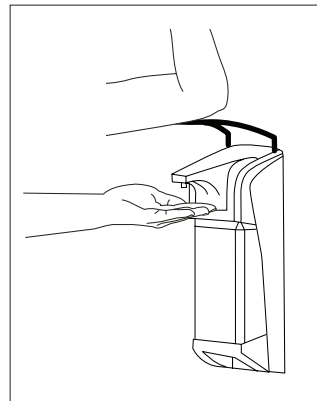
E Mounting

Dispenser system with locking kit:
Remove locking kit and then the container.

Dispenser system without locking kit:
Remove the container.

Slightly press the fixing catch and pull down the wall plate. Fix wall plate with enclosed plugs and screws. Mount the dispenser onto the wall plate by firmly pressing until it clicks into place.

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
E Operation

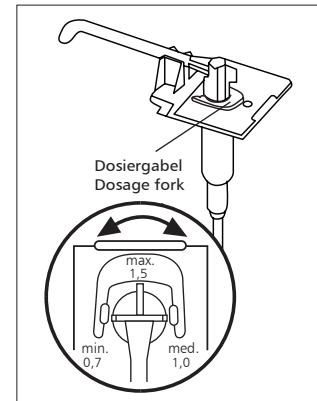For initial working, the unit must be operated several times until it delivers fully. Hold the open hand under the outlet nozzle.

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
E Dosage

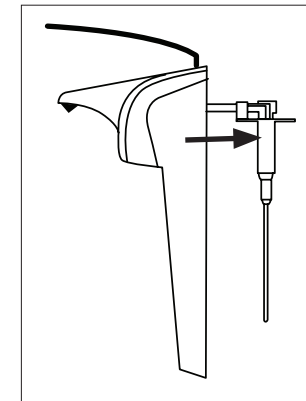The dispenser is adjusted to the maximum amount of approx. 1.5 ml/stroke. If required, the dosage can be reduced to approx. 1.0 ml/stroke or 0.7 ml/stroke by gradually turning the dosage fork.

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E Pump exchange

Hold operating lever up and pull the pump out. For inserting the pump hold operating lever up and place pump into position.