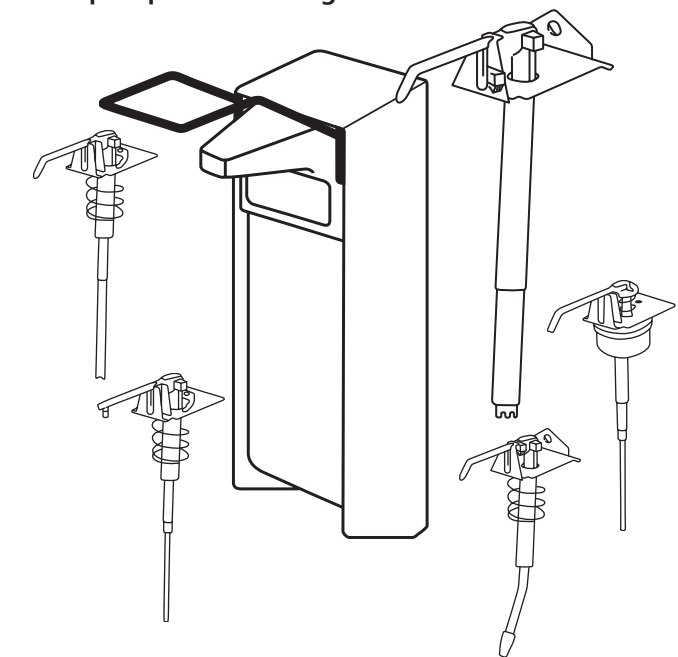
ingo-man® 26

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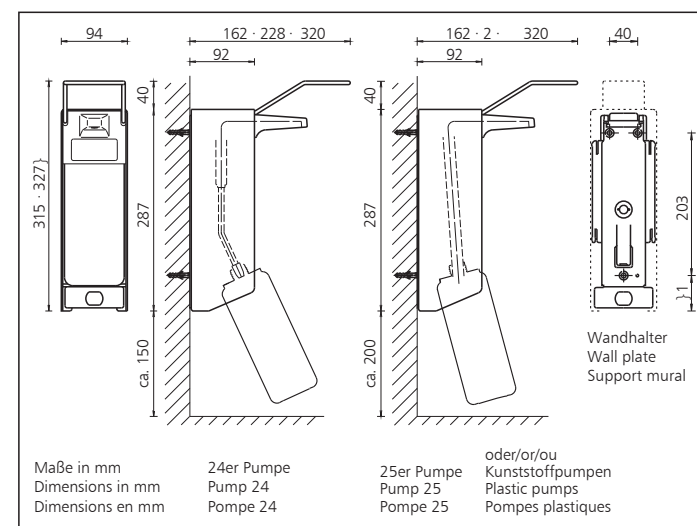
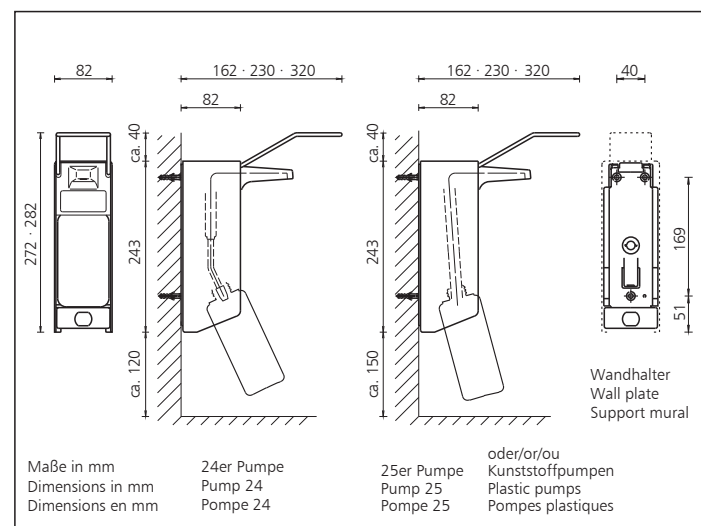
Dispenser for soap and disinfectant with interchangeable pump units

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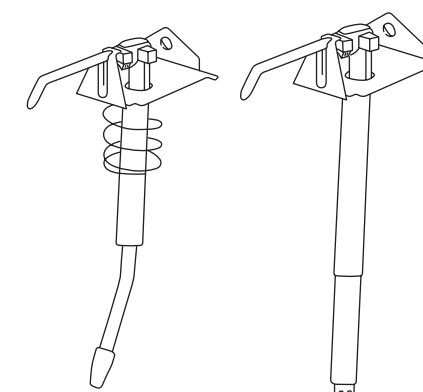
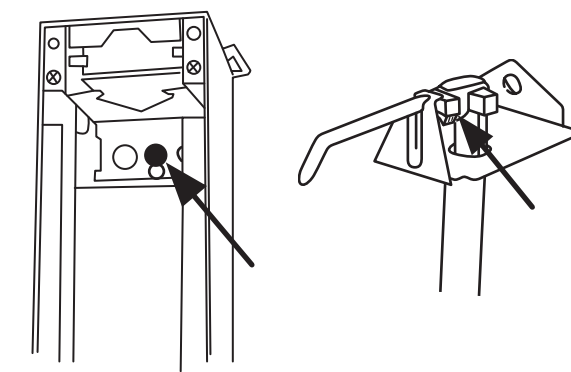
Pump type 24 with crooked stainless steel sucking tube, autoclavable

Pump type 25 with straight stainless steel sucking tube,

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Each unit is colour coded inside of the chassis. The pumps are marked by a coloured dosage wheel (except foam pump, see below). The colour indicated on the base unit must be same as of the dosage wheel.

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The compatibility of filling material and dispenser system must be checked prior to the use of new filling materials, in order to be able to guarantee fault-free functionality. OPHARDT Hygiene assumes no liability for any damages or costs which arise due to the incompatibility of filling material and dispenser systems.

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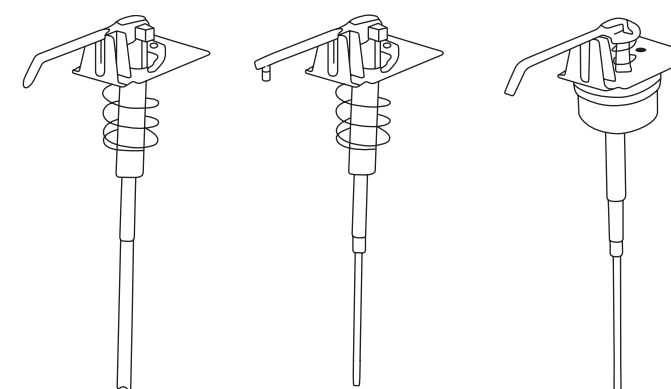
Plastic pump with straight plastic sucking tube, non autoclavable

Spray pump for disinfectant, plastic non autoclavable

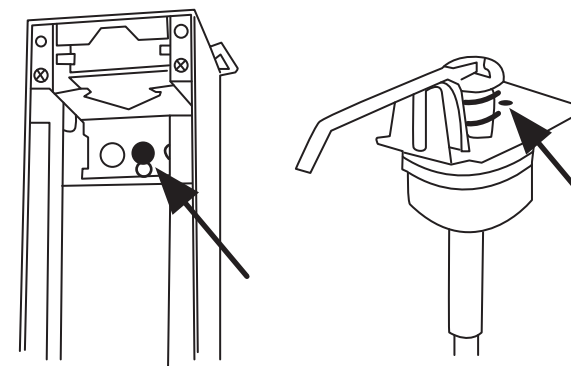
Pump for foamy soap, plastic, non autoclavable

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Each unit is colour coded inside of the chassis, also the foam pump. The colour indicated on the base unit must be same as on the foam pump.

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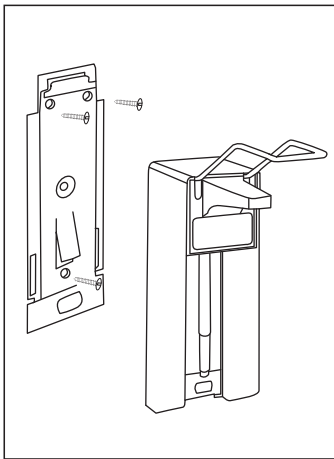
Check before installation

1. The dispenser should be fitted above a sink so that wet hands can drip.
2. Required space above the dispenser is 40 mm; for below refer to dimensions overleaf.
3. Required height: 130–135 cm, from floor to top of dispenser.

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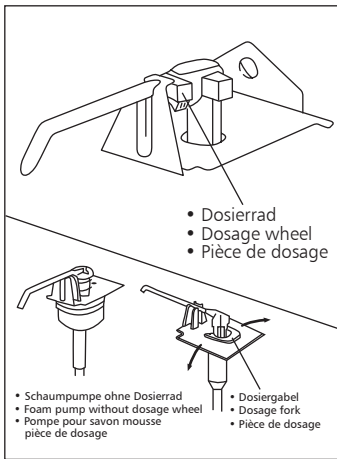
Mounting

Remove the container. Slightly press the fixing catch and pull down the wall plate. Fix wall plate with enclosed plugs and screws. Firmly press the dispenser on to the wall plate until it clicks into place.

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Dosage

The dispenser is adjusted to the maximum amount of approx. 1.5 ml/stroke. If required, the dosage can be reduced to a minimum of approx. 0.5 ml/stroke by gradually turning the dosage wheel head.

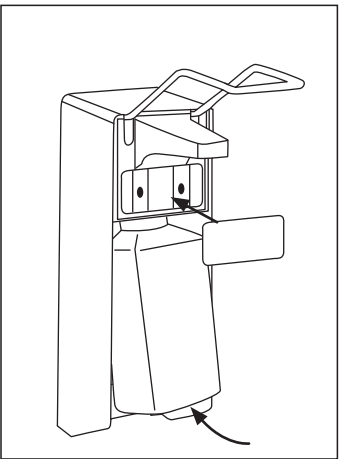
The plastic pump is adjusted to the maximum amount of approx. 1.5ml/stroke. If required, the dosage can be reduced to a minimum of approx. 0.8 ml/stroke by gradually turning the dosage fork head. **The foam pump has not a dosage wheel. The dosage is not changeable.**

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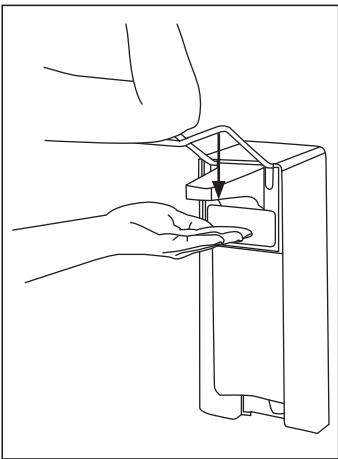
Container exchange

Insert the refilled container from the bottom and press the facial clip into position.

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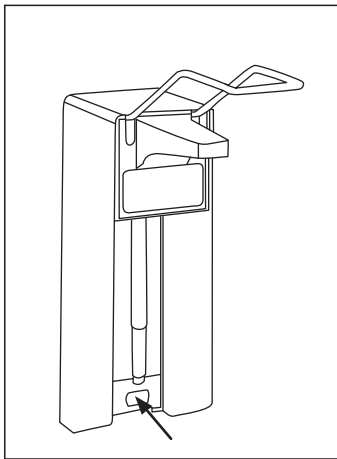
Operation

For initial working, the unit must be operated several times to secure the correct dosage. Hold the open hand under the outlet nozzle.

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Cleaning/Sterilizing

Press locking device and push dispenser upwards. Dispensers made of aluminum or stainless steel and stainless steel pumps (also pump type 28 with flexible sucking tube) are autoclavable up to 121 °C, 2 Bar.

Plastic pumps and powder coated dispensers are not autoclavable!

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Maintenance

The Robert Koch-Institute recommends: "Dispensers for lotion and disinfectants have to be easily cleaned and disinfected. Lotion dispensers must be thoroughly cleaned and disinfected whenever cartridges are changed." [Source: Robert Koch-Institute, recommendations for hand hygiene]. In order to guarantee proper functioning of the dispenser regular manual cleaning or processing in cleaning-disinfection devices is necessary. Remove the dispenser from the wall plate: to do this, press the latch on the back wall of the dispenser and slide the dispenser upwards, off the plate. Manual cleaning: Regularly remove adherent product residual. Cleaning of the dispenser housing and the pump with hot water. Rinsing the pump with hot water. Disinfection of shroud is possible with appropriate surface disinfectants. Processing in cleaning-disinfection devices: choose program „Dispenser cleaning“ and use pH-neutral cleaning solutions. Processing in autoclave: the dispenser and stainless steel pump are autoclavable with superheated steam (121 °C, 2 bar, max. 20 min dwell time). Plastic wall plate and bottle are not autoclavable.

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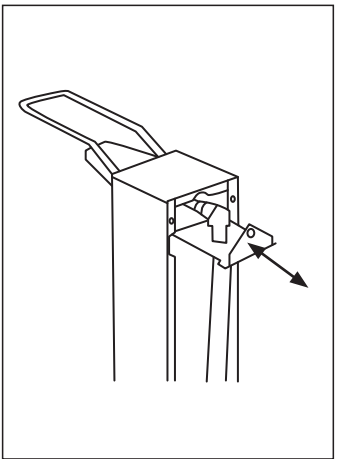
Pump exchange

Make sure that the colour code inside the chassis matches with the colour of the dosage wheel.

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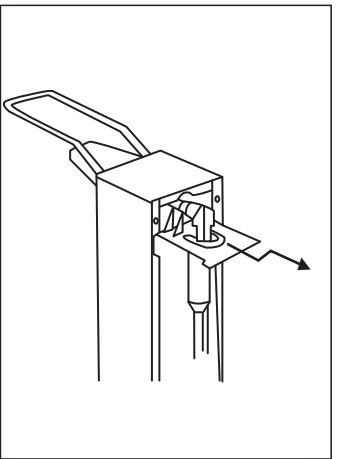
Exchange of stainless steel pump

Hold operating lever up and pull out the stainless steel pump from the back. For inserting the stainless steel pump hold operating lever up and place pump into position.

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Exchange of plastic pump

Hold operating lever up, slightly lift the plastic pump and pull it out. For inserting the plastic pump, hold operating lever up and place pump into position.

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