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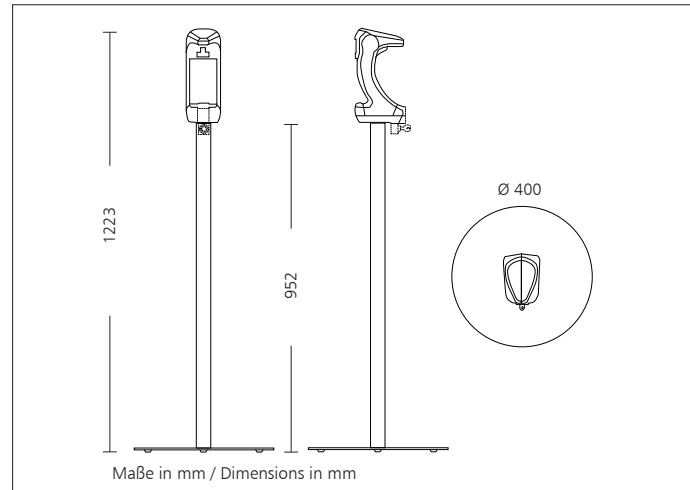
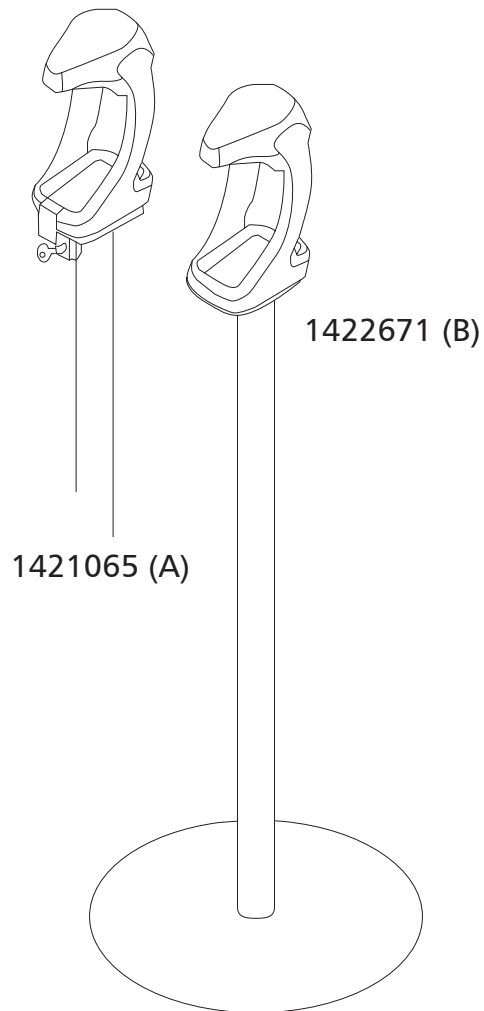
RX 5 T stand Art.-No. 1421065 (with lock, A), 1422671 (without lock, B)

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Safety information - Attention

- Mount the column vertically on a flat level surface.
- Do not install the appliance in passageways!
- Tighten all fittings.
- Not a toy! Use only under adult supervision.
- Use only the OPHARDT Hygiene RX 5 T for the column.
- Maintenance: Please check regularly to ensure the column is stable. Tighten any loose fixings.

Maintenance

- Wipe column with a lightly moistened cloth to remove product residue
- Please do not use acidic, caustic (chlorous) purifiers or scouring powder.
- Use only disinfectant that is compatible with the material

Mounting information

- Please remove the protective film before mounting!

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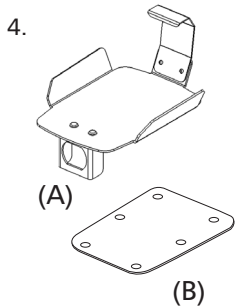
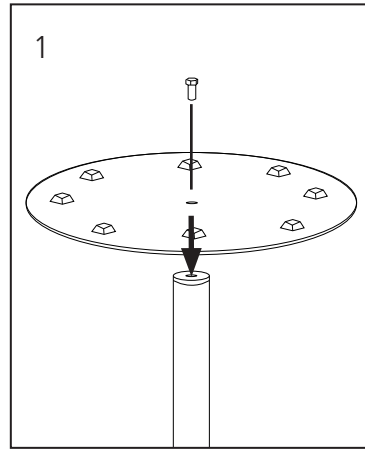
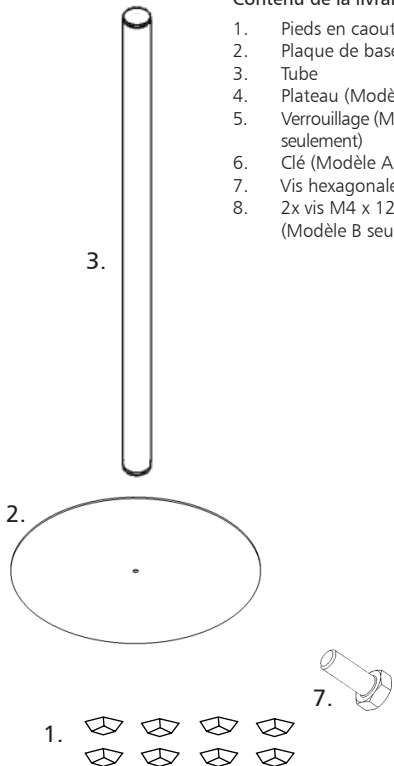
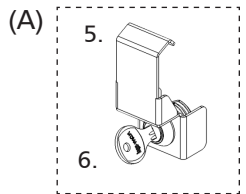
Scope of Delivery:

1. Rubber feet
2. Base plate
3. Tube
4. Tray - Model A and B
5. Lock (only Model A)
6. Key (only Model A)
7. Hexagonal M8 screw
8. 2x M4 screws x 12 (only Model B)

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
Remove the adhesive backing from the rubber feet (1.) and stick them to the base plate (2.) max. 2 cm from the edge (space evenly). Connect the base plate (2) with the tube (3.), with the upper side of the base plate facing the tube. Fasten from the underside of the ground plate with M8 screw (tool required).

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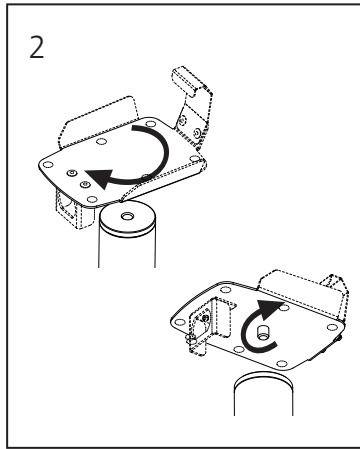
2. Turn the stand upside down and place it on the ground plate. Screw the dispenser tray (4.) onto the tube and tighten hand-tight.

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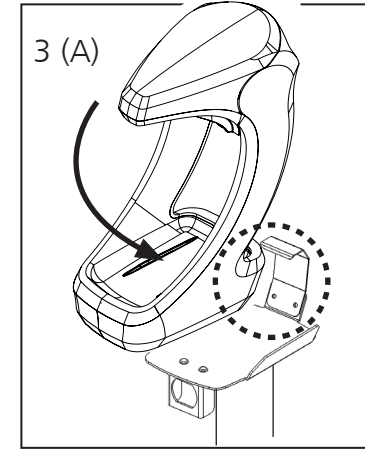
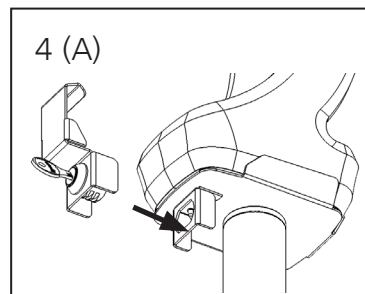
4. Afterwards fix the lock for safety mounting to the dispenser. Close with the key.

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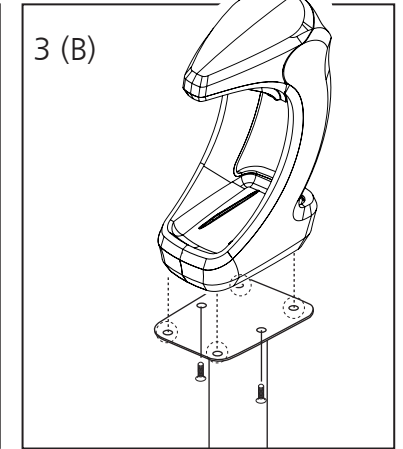3. Screw on the dispenser from below with 2 screws provided: Make sure that the dispenser feet are located in the 4 holes in the tray.

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